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## Instructions for use 2-piece ostomy system Combimate, Combiplast and Ostomate II drainable pouches

|  |
|--|
| <b>General information</b>   |
| <b>The pouches</b> <p>The pouches can have an inspection window, making them easier to apply and allowing inspection of the stoma from time to time without removing the pouch <b>(A)</b>. The drainable pouches are equipped with various closure systems: Velcro, wire, bottom tap or clip closure.</p>  |
| <b>The filter (if applicable)</b> <p>Drainable pouches are equipped with a filter. To seal the filter off completely, for example when having a bath or when swimming, use the supplied filter stickers. These pouches also have an additional foil layer inside the pouch to protect the filter. This prevents most leakage through the filter. The drainable pouch is by default fitted with a filter membrane to prevent leaking through or staining clothing</p> |

**The belt connection**

The belt connection is made to be used with the EuroTec wide belt with 2 hooks.

**Intended use**

Drainable pouches are intended to be used by ostomates, to collect their (thin) stool. Using this product does not normally require the aid of a medical professional. Drainable Pouches are intended to be used in conjunction with compatible skin wafers. Combimate, Combiplast and Ostomate II wafers.

**Indications**

- To be used for collecting (thin) stool.
- To be used in combination with a 2-piece system wafer.

**Contra-indications**

There are no contraindications for the use of the drainable pouches.

**Warnings and precautions**

All pouches are for single-use only and should not be re-used. Re-use may lead to increased risk of infection and cross contamination. Physical properties of the device may no longer be optimal for intended use. Disposal of the pouches should be done through normal household waste.

|   |
|---|
| <b>Instructions</b>   |
| <b>Preparation</b> <p>Ensure you have all necessary items at hand.</p>  |
| <b>The application of the pouch</b> <p>Start at the underside of the flange ring and with two fingers push the pouch onto the flange over the complete circumference. Continue over the left and right side, until you feel a final click. Always check if the pouch is properly fixed, by pulling at the pouch. A pouch should be worn for no longer than 72 hours before removal.</p> |
| <b>The removal of the pouch</b> <p>Support the skin wafer and carefully pull the tab on the top side of the flange forward <b>(B)</b>. Clean the stoma and the area inside the gasket of the flange with lukewarm water before applying a new pouch.</p>  |

|   |
|---|
| <b>Opening, closing and emptying of the pouch</b> <p><b>Velcro closure:</b></p> <p>The drainable pouches have a SOFTSAFE fastening with a ‘finger pocket’. The fastening can be opened by unfastening the Velcro and rolling it open. To open the bottom of the pouch, put one finger in the sleeve (the ‘finger pocket’) <b>(M)</b> and two fingers to the sides of the pouch. Use the other hand to clean approximately 1 cm of the inside of the pouch. The fastening can be closed easily by folding the end 5 times <b>(K</b> and <b>L)</b>.</p> |
| <b>Wire clip:</b> <p>A wire clip can be rolled up, and the wire ends folded inwards to close and secure the tail <b>(F)</b>.</p>  |

**Bottom tap:**

High Output pouches have a drainage tap with a soft plug **(G)**, that can be connected to a drainage bag by means of an adapter **(H)**. The plug needs to be fully inserted into the tap **(I)**. If desired, you can cut off the drain tap and close the pouch with the clip provided **(J)**.

**Clip closure:**

Fold the tail of the pouch over the narrow part of the clip **(K)**. Avoid creating any folds. The clip can be closed by snapping the two sections into each other **(L)**. When the clip is correctly attached, the concave side must lie against the side of the body. The clip can be opened by unsnapping the two sections.

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